

Maternity Simulation Jacket

with storage bag

Maternity Simulation Jacket II

Outline

This simulation jacket is ideal for experiencing the life and activities of a pregnant woman, and learning how to assist her. It will cater especially to the educational needs of midwifery, public health nursing, nursing and also for expectant parents.



LM-054



LM-065

Features


- The jacket is designed to fit an average woman of 155-165 cm in height, but also will fit an average man of up to 185 cm in height.
- Quilted material is used for the jacket to allow close contact with the body.
- Adjustable straps allow the jacket to be worn by people of a variety of different sizes.
- A high safety standard is assured thanks to a urethane gel filling and weight adjustment bags containing iron particles.
- The weights can be adjusted and fixed with hook-and-loop fasteners to enable simulation from second to third trimester pregnancy.
- Abdominal fetus parts and amniotic fluids are made from urethane gel and give a sensation very close to that of a real human body.




Maternity clothing is not attached.

Name of Each Part

Maternity Simulation Jacket (LM-054)

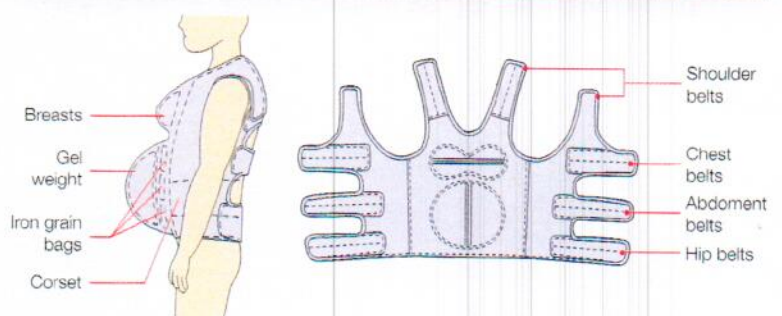

Maternity Simulation Jacket II (LM-065)

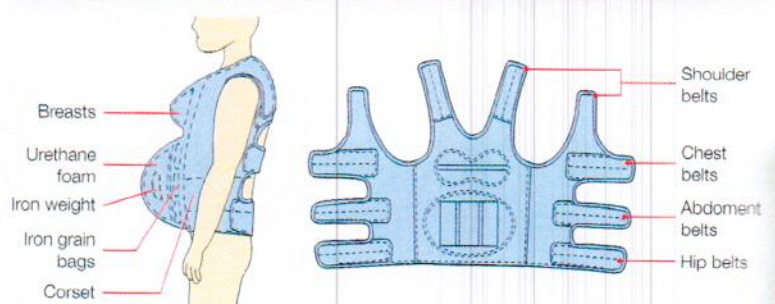

■ How to Wear



- 1 Wear the corset without fail to prevent lumbago.



- 2 Set the abdomen of the maternity simulation jacket to the abdomen of wearer.



- 3 Adjust the abdominal position by using the shoulder belts.



- 4 Fix temporarily. Always start from the shoulders and proceed to chest, abdomen and hip belts.



- 5 Tighten the belts of the maternity simulation jacket so that the jacket fits snugly to body. (If the jacket does not fit snugly, it can cause lumbago and it will be difficult to accurately experience the feeling of pregnancy.)



■ Washing Instructions

Always remove the breasts, the gel weight and the bags containing lead weights when washing the jacket. Hand wash only in lukewarm water at temperature not exceeding 30°C. Do not use chlorine detergent. Iron the jacket with a medium temperature iron, using a cloth.

■ Storage Instructions

Store in the bag with the abdominal side on top. If this side is on the bottom, the abdomen may become deformed due to the lead weights.

■ Accessory

Corset for preventing lumbago

Specifications

LM-054

Material	10 % cotton, 90% polyester
Weight	Approx.7.3 kg

LM-065

Material	100 % cotton
Weight	Approx.7.3 kg

Spares

LM-06012	Storage bag	1
----------	-------------	---